

Indication:

This device is intended for the management of:
Burns, Sores, Blister, Scrapes, Ulcers, and other Wounds

Product Application:









- Cleanse the wound per physician order.
- Apply CONNECT-CSM particles ¼ inch thick to the entire wound surface.
 - Do not pack tightly; allow for expansion of particles.
- Cover with an absorbent dressing.
- Use CONNECT-CSM product until the wound is fully healed.
- Maintain a moist wound environment.

Product Removal:

- Change as indicated by the amount of drainage and instructions provided with the absorbent cover dressing.
- After removal of the cover dressing, repeat steps under
- Product Application as needed.

Precautions:

- This product is made from bovine (cattle) derived materials.
- Do NOT use this product if bovine-derived product such as leather cause an allergy reaction or sensitivity upon contact with the patient's skin.
- WARNING: Discontinue use of CONNECT-CSM particles and notify your doctor if excessive redness, pain, swelling, or blistering occurs.
- WARNING: Potential risk of loss of sterility and infection if opened and reused.
- This device is intended for single use only. All open and unused vials must be discarded.
- If sensitivity develops, discontinue use.

STERILE R	Method of sterilization: Irradiation
	Do not reuse
	Do not resterilize
	Keep dry
	Do not use if package is damaged
	Keep away from sunlight
	Not made with natural rubber latex
	Temperature limitation
Rx Only	Prescription use only
	Instruction for use in electronic form



Orthonovis Inc.
1 Hargrove Grade, 2f
Palm Coast, FL 32137

cs@orthonovis.com
833-950-0127

